# BD125X10/2

## Indicador Químico

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### Uso previsto

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*Bowie & Dick Test Pack*

### Usage

Bowie & Dick test detects inadequate air removal and steam penetration in vacuum-assisted Steam sterilizers. Cycles of 10 minutes at 121 °C.

### Applicable Regulation

ISO 11140-1:2014 and ISO 11140-4:2007

### Product Classification

Class 1, according to risk.

### Authorization

Designed under Quality Management Systems ISO 9001:2008 and ISO 13485:2012.

### Characteristics

Type 2 Chemical Indicator

Initial Color: yellow

Final Color: dark brown/black (should change homogeneously in the whole sheet)


Size: 16 mm x 125 mm x 125 mm

Assembly: 22 permeable barriers - 1 Glassine paper - 1 B&D Test Sheet - 22 permeable barriers. Wrapped with crepe paper.

Permeable barriers: 200 g/m<sup>2</sup>.

B&D Test Sheet: 240 g/m<sup>2</sup>.

Weight per unit: 157 g.

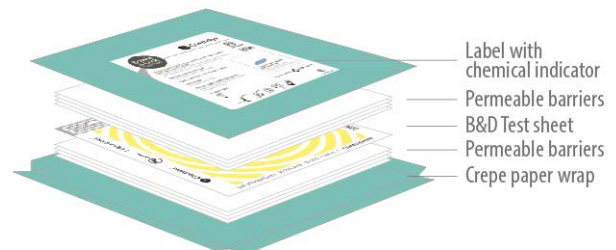### NOTE:

Air removal or steam quality problems will be shown as heterogeneous color change into dark brown/black on the test sheet, such as lighter areas in the center or a non-uniform color change.

100 % Lead Free

### Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

**Storage conditions**

T = 10-30 °C, RH 30-80 %, keep out of direct light.

**Transportation conditions**

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

**Shelf-life**

5 years

**Packing**

20 units per box. Weight per box: 3270 g

Packing information: product code and description, process for intended use, classification according to regulation, manufacturer information and data on label.

**Labelling**

On product: label printed with Type1 indicator ink, turns from blue to dark grey/black. Not exposed: blue, exposed: dark grey/black. Product code and description, process for intended use, classification according to regulation, batch number and expiration date.

On product's box: Product code and description, process for intended use, classification according to regulation, batch number and expiration date.

**Possible target markets**

Healthcare, Food, Pharmaceutical and Medical Industries.

**Other important information (optional)**

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

**Description of identified and/or predictable risks which could represent a risk for the product**

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization process other than Steam.

Do not use a Bowie& Dick Test Pack in gravity displacement steam sterilization cycles (where there is no pre-vacuum phase to check) or any kind of sterilization processes other than vacuum-assisted Steam.

Do not reuse the sterilizer until the chemical indicator has changed homogeneously into the color indicated in the product.