CDWA4

Indicador de lavado

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Uso previsto

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For cleaning performance tests.

Usage

Designed for cleaning process routine control and monitoring automatic washing machines effectiveness as well as monitoring ultrasonic washers which operate at frequencies equal or greater than 35 kHz, and temperatures between 20 °C - 65 °C.

Applicable Regulation

ISO/TS 15883-5:2005

Authorization

Designed under Quality Management Systems standards ISO 9001:2008 and ISO 13485:2012.

Characteristics

Challenge level: very high

Allows detection of all factors affecting the outcome of cleaning process

Initial Color: fuchsia

Size: 70 mm x 30 mm

Synthetic temperature-stable support.

Result: elimination of fuchsia organic substance.

NOTE: If unfavorable results are obtained, check quality/quantity of detergent, cleaning time and temperature, positioning of the load, CDWA4 indicator location and general condition of the washing machine.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 10-30 °C, RH 30-80 %, store in dark place.

Transportation conditions

Storage conditions should be strictly followed.

Transport in closed and strengthened boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

5 years

Packing

200 units per aluminium foil bag.

Packing information: product code and description, process for intended use, classification according to regulation, operative instructions, manufacturer information and data on label.

Weight per bag: 165,7 g

Labelling

On product's box: Product code and description, process for intended use, classification according to regulation, batch number and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information (optional)

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Store the product within its' original package, away from light, disinfectants, gases and humidity.

Do not re-use the product.

Do not touch indicator ink with your hands.